

INOGEN ONE G2 - Introduction



June 2010

What lead to INOGEN ONE G2?

1. INOGEN ONE since 2006

- Focussed on patient's needs
- Technically optimised

2. Additional product goals:

- Lighter in weight
- Smaller in size
- Better performance (Battery'O₂-flow)

The Changes

	INOGEN ONE G1	INOGEN ONE G2
Size	31 x 29 x 15 cm	24 x 27 x 10 cm
Volume	9000 cm ³	5200 cm³
Weight	4,5 kg	3,2 kg
Noise level	< 39 dbA	< 38 dbA
12 Cell Battery	2,5 - 3 hr.*	3,5 - 4 hr.*
24 Cell Battery	N/A	7 - 8 hr.*
O2 Volume	Max. 750 ml**	Max. 900 ml**

* At Setting 2

** At Setting 5

How quiet is the INOGEN ONE G2?

Level of hearing	0 dBa
Wind through leaves	20 dBa
Whisper (1 mtr)	30 dBa
INOGEN ONE G2 (Setting 2)	35 dBa
INOGEN ONE G2 (Setting 5)	38 dBa
At home, quiet	40 dBa
Street, quiet	50 dBa
Normale conversation	60 dBa
Car, inside	70 dBa
Loud singing	75 dBa
Lawn mower	107 dBa

Next to each other



Battery Specifications

INOGEN ONE G2	Run Time* (hr.)	Charging (hr.)	Weight (Kg.)
12 Cell Battery (BA-200)	3,5 – 4	3 - 4	0,7
24 Cell Battery (BA-224)	7 - 8	6 - 8	1,3

* At Setting 2



With 12 Cell-Battery the
INOGEN ONE G2 weight
is 3,2 kg



The Batterie slides under the
Concentrator



With 24 Cell-Battery the
INOGEN ONE G2 weight
is 3,8 kg

The Benefits

Small
Low Weight
24 Cell Battery
Low Noise level
Display in local language
No Repairs
Simple operation
Reliability
Battery life
Little maintenance
Suited for night time use
7.5 meter cannula
Universal Power Supply

Maintenance

To be performed by the patient:

- Clean Particle Filter on weekly basis (replace Particle Filter every 3-4 months).
- Battery Care/ first time use and maintenance:
 - Fully charge battery prior to first use (3-4 hrs);
 - When not using Concentrator for longer periods (> 2 weeks) => remove battery from concentrator
 - Once a month fully discharge battery and fully charge it afterwards
- Replace Output Filter once every 8000 hrs of operation or once a year (by INOGEN ONE Specialist).

Warranty Information

INOGEN ONE G2 System:

- INOGEN ONE G2 concentrator => 3 years
- Battery, Caddy, Carry Bag and Universal Power Supply => 1 year



What benefits does INOGEN ONE G2 offer?

To the user:

- Quiet => it is so quiet one will hardly notice INOGEN ONE G2 is running
- Small => INOGEN ONE G2 will stay unnoticed at home or away from home
- Independence => the user will regain flexibility to live the desired life; an increase in quality of life
- Performance => little energy consumption resulting in long battery life (12 Cell or 24 Cell)
- User-friendliness => every patient can operate and use INOGEN ONE G2.

Availability INOGEN ONE G2

What do we offer?:

- The INOGEN ONE G2 Programm is available since April 2010
- Sales and advice through preferred Equipment Provider specialized in long term Oxygen Therapy
- Please find an Equipment Provider in your country (Information, Delivery, Wartung) or near to you. Please find an updated listing on our web bsite www.inogenone.eu or send an e-mail to info@inogenone.eu.

INOGEN ONE^{G2}



Thank you!!

Questions? Call +31 346 216985 or E-Mail: info@inogenone.eu